EN	Instruction For Use Procedure Pack To Aid Infusion	02	>
DE AT	Gebrauchsanweisung Behandlungseinheit: Infusions-/Injektionszubehör-Set	03	>
BE NL	Gebruiksaanwijzing Verpakking met infusiehulpmiddelen	04	>
BG	Инструкции за употреба Помощен пакет при вливания	05	>
HR	Upute za uporabu Komplet za pomoć pri infuziji	06	>
CZ	Návod k použití Pomocná sada pro infuzi	07	>
DK	Brugsanvisning Administrationssæt til infusion	08	>
FI	Käyttöohjeet Toimenpidepakkaus infuusion avuksi	09	>
FR	Mode d'emploi Nécessaire d'aide à l'injection pour faciliter la perfusion	10	>
HU	Használati útmutató Kiegészítő készlet infúziós alkalmazáshoz	11	>
IT	Istruzioni per l'uso Set di Infusione	12	>
LT	Naudojimo instrukcija Rinkinys infuzijos procedūrai	13	>
NO	Bruksanvisning Administreringssett til infusjon	14	>
SI	Navodila za uporabo Komplet pripomočkov za infundiranje	15	>
ES	Instrucciones de uso Kit para procedimientos para facilitar la infusión	16	>
SE	Bruksanvisning Administreringsset för att underlätta infusion	17	>

Instruction For Use

Procedure Pack To Aid Infusion

INTENDED USE

The procedure pack should be used to support infusion or injection activities. It consists of:

- 2x pre-injection cleaning swab, saturated with 70%v/v isopropyl alcohol, B.Braun REF 19579 – used to clean the skin before infusion or injection.
- 2x sterile gauze swab, Paso-Trading, REF CH-001485, CE 2274 – used as a physical barrier to absorb small amounts of blood, wound exudate and to protect the wound from infection.
- 2x sterile adhesive surgical dressing, Paso-Trading, REF CH-001476, CE 2274 – used for direct protection of the skin or fixing sterile gauze swab after infusion/injection procedure.

The sterile gauze swab and sterile adhesive surgical dressing are suitable for the skin contact no longer than 24 hours.

INTENDED USER

Trained patients or caregivers (parents, siblings, or partners) in a home use environment, or by trained Healthcare professionals in an office/ hospital setting.

PRECAUTIONS

1. Carefully follow the instructions for use. 2. All components shall be used directly after the individual direct package is opened. 3. Don't apply for wounds heavily bleeding. 4. Don't use if individual package is damaged. 5. Don't exceed 24h of the skin contact.

STORAGE

Store the Procedure Pack in a dry room and away from the sunlight.

DIRECTIONS FOR USE

1. The cleaning swab (REF 19579) should be placed just above the intended injection site, on the intact skin and should then be drawn across the injection site only once and in one direction. The skin should then be allowed to dry naturally prior to injection.
2. Complete the injection according to right IFU connected with the medicinal product.
3. Protect the injection site with the sterile gauze (REF CH-001485) and fix it to the skin with the adhesive surgical dressing (REF CH-001476).

WARNINGS

1. This product is for single use only; reuse may cause a risk of infection. 2. Do not re-sterilize. 3. Do not re-pack. 4. Not suitable for any other application than intended use. 5. The alcohol swab can be applied only on the intact skin. 6. The manufacturer cannot be held responsible for any unauthorized or unqualified use of the device.

WASTEDISPOSAL

Unused products may be considered as non-hazardous waste and disposed accordingly. Dispose of used products as well as any other contaminated disposable materials following procedures for infectious or potentially infectious products.

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